#### Istruzioni:

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13-5009 REV A

## EN

Chatt-tape is an elastic, adhesive tape without medicines and is made of high-quality cotton with an acrylic adhesive layer. Chatt-tape can be worn for several days, is free from latex and is water resistant.

Chatt-tape may only be used by professional users who have had training in the special kinesiology-based taping technique.

### Directions for use:

- In rare cases, the use of Chatt-tape can cause skin irritations or allergic reactions. The treatment should be stopped immediately if adverse reactions occur and the tape should be removed from the skin.
- It is recommended that the user first applies a small piece of tape if he/she is over-sensitive to the use of plasters or tape. It must then be borne in mind that allergic reactions do not always occur immediately, but that a later reaction (after approximately 24-48 hours) can also occur. Remove the product immediately in case of adverse reactions.
- Do not use Chatt-tape on wounds, burns and mucous membranes, nor on the mouth, nose or eyes.
- The skin should be cleaned before applying the Chatt-tape.
- When removing the Chatt-tape, the tape should be removed slowly in the direction of the hair growth. If the removal of the tape proves difficult, it is advised to use a tape removal agent.

### Instructions:

- Keep Chatt-tape at room temperature (between 4 - 25 °C) and not in direct sunlight.
- Do not put Chatt-tape on or near radiators or any other source of direct heat.
- Keep the tape out of reach of children.
- For single use only.

• Injuries can always occur, even when Chatt-tape is used in the proper manner.

**DJO Global, Inc. / THYSOL Group BV are not liable for any injuries and/or damage due to Chatt-tape or the unprofessional use of the product.**

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©2015 DJO, LLC