AIRGO SPRUNGGELENKORTHESE
 AIRGO ANKLE BRACE
 AIRCAST®

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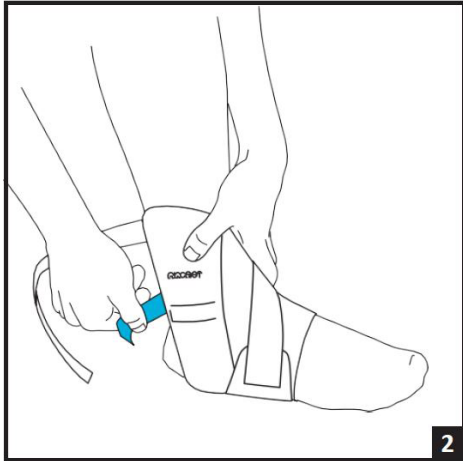
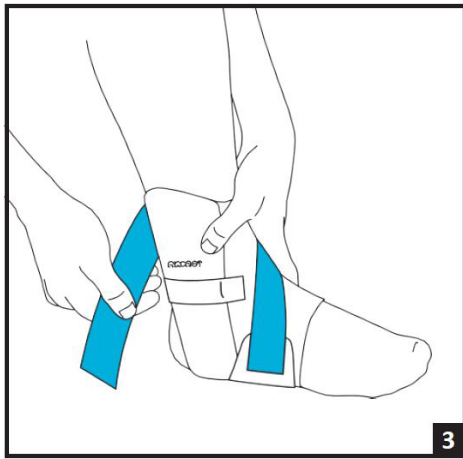
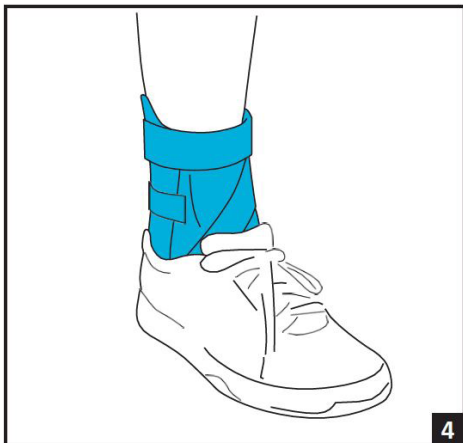
4

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: The AIRCAST AIRGO ankle orthosis is designed to support and stabilize the ankle joint following acute ligament ruptures, severe distortions and chronic ligament instabilities. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND CAUTIONS: This product should be worn only according to the doctor's prescription or instructions by healthcare professionals. If pain, swelling, changes in sensation, or unusual reactions occur during the use, the attending physician should be consulted immediately. In case of the following symptoms, the orthosis should be used only with the expressive consent of the doctor:

- Lymphatic drainage disorders
- Circulatory disorders
- Sensory disturbances
- Skin diseases and irritations
- Open wounds
- Lymphedema
- Do not use this device if it is damaged and/or package has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

DIRECTIONS FOR USE:

- Please ensure that the Velcro does not damage the clothes.
- Please put on a stocking.
- Loosen the straps and open the orthosis.
- Put your foot into the orthosis from behind, with your toes first (Fig. 1). Align the side stabilizers with the ankle.
- Guide the small strap around the back of the ankle, tighten it and fix it on the side (Fig. 2).
- Tighten the diagonal ATF strap until it is firmly in place. Tie the drawstring around the leg, tighten it and fix it (Fig. 3).
- Put on a support shoe or sports shoe (Fig. 4).
- To align the orthosis finely, you can tighten or loosen the ATF drawstring.

CLEANING INSTRUCTIONS: The AIRGO ankle orthosis can be hand washed in 30° with a mild soapy solution and then air dried.

MATERIAL CONTENTS:

- Outside: Polyamide, Polyester, PVC
- Inner lining: Polyamide, Neoprene
- Velcro straps: Polyamide
- Air chambers: PVC, Polyether foam

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.