## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:** The DonJoy Back Brace TSO is designed to provide comfort and postural support to the spine. It can be utilised for pre- and post-surgical care. It may be suitable for a variety of thoracic spinal abnormalities and the symptomatic relief of acute and chronic back pain.

**Performance Characteristics:**  
Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.  
**CONTRAINDICATIONS:** Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which pose risk to be made worse as a result of compression of the abdomen and/or pressure. If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

**WARNINGS AND PRECAUTIONS:**  
• This product is to be fitted and prescribed by a physician.  
• For maximum efficacy, ensure the correct size of support is selected and the side pulls are adjusted to the required fit.  
• Pain, numbness or any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.  
• Do not use over open wounds.  
• Always inspect product prior to application to ensure all parts are in proper working order.  
• Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**APPLICATION INFORMATION:**  
**Prepare Brace:**

- A. Open both of the anterior/side Velcro® panels. Keep the shoulder straps attached and extend to maximum length. (Fig. A)

- Application of Brace:**
- With patient standing, lift the TSO over the patient's head. Lower the TSO into position keeping the patient's head between the two shoulder straps. Let the front/anterior panel and back/posterior panel hang over the torso.
  - Close both of the side panels so that they are even and comfortably snug on the Anterior panel (Fig. 1).
  - Adjust the sternal Y-Bar while the patient is seated in prescribed posture. Pull up on the black release button (on top portion of anterior panel) to release the sternal bar lock. Slide the sternal Y-Bar into position, so that the top of the sternal pad fits 1/2" below the sternal notch. Be sure to keep the bottom edge of the Anterior Panel aligned with the pubic symphysis (Fig. 2).
  - To adjust the shoulder straps, release and adjust the posterior Velcro® straps first. Adjust to desired length. Be sure to maintain the bottom edge of front panel with the pubic bone and the back panel between shoulder blades and lower back (Fig. 3).
  - To activate support, grasp push-handle and pull straight out from the torso. After the desired level of support has been achieved, fasten the pull handle to anterior portion of the support panels. Adjust support after five minutes, then re-adjusting as needed (Fig. 4).

**Removal of Brace:**

- Release the handles and remove one of the shoulder straps.
- Release one side of the side panel from front panel and slide the orthosis off the other shoulder like removing a jacket or backpack.

**CUSTOMIZING TSO: (To be performed by a licensed medical professional only)**

- **Waist Belt:** If needed, to modify the waist belt, adjust the side panels by un-hooking the Velcro on the back panel and moving panels more center for a smaller waist, or closer to the edge of the back panel for a larger waist.
- **Shoulder Straps:** If needed, the extra length in the straps can be trimmed with scissors.
- **Sternal Bar:** May be bent as needed by using a stable edge of a table. Using both hands, bend the bar in a slow and controlled fashion to adjust to desired degree of offset. The stock sternal Y-bar is set at 15°.


**USE AND CARE (CLEANING & STORAGE INSTRUCTIONS):** Hand wash in cold water, less than 86°F (30°C), using mild soap. Air dry.

**MATERIALS: STERNAL BAR:** Aluminum, Nylon, Polyurethane; **WAIST BELT:** Polyester, Polyurethane, Nylon, Polypropylene; **SHOULDER STRAPS:** Nylon, Polyester, Polyurethane.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations; the provisions of such local regulations will apply.

**RX ONLY**

**NOT MADE WITH NATURAL RUBBER LATEX**

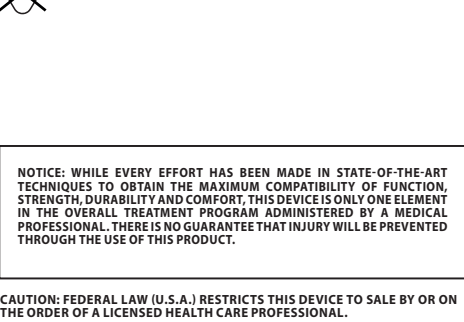**CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROVIDER.**

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