**ENGLISH**

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:**

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INDICATIONS:**

The Aircast Diabetic Replacement Kit contains a replacement insole and stocking for use with the Pneumatic Walker Diabetic System. Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:** NA**APPLICATION INFORMATION FOR PINK PLASTAZOTE INSOLE:**

- 1) Remove front panel. Open foot flaps and toe cover.
- 2) Firmly grasp edge of insole and remove from brace (Fig. 1). Grey insole remains in bottom.
- 3) Remove backing from tape on bottom of new pink insole and firmly press on top of grey insole.

**APPLICATION INFORMATION FOR IMPAX GRID INSOLE:**

- 1) Remove front panel. Open foot flaps and toe cover.
- 2) Firmly grasp edge of insole and remove from brace (Fig. 2a). Grey insole remains in brace.
- 3) Locate and remove the pre-cut foam cubes from the bottom of the pad around the wound site/s (Fig. 2b). Remove the pink Plasztazote Insole that comes in the walker and replace with the Impax Grid Insole (pink side down). Blue side of Impax Grid Insole should be against plantar side of foot.

**WARNINGS AND PRECAUTIONS:**

- This device is to be used under the supervision of a healthcare professional. The determination of when to apply the brace and the frequency and duration of use should be strictly at the discretion of the treating physician. Always consult with your physician before making changes to the brace.
  - Do not use over open wounds.
  - Do not modify the device or use the device other than intended.
  - Do not use this device if it is damaged and/or the packaging has been opened.
  - Do not use this device on patient's incapable of communicating physical discomfort.
  - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
  - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**CLEANING INSTRUCTIONS:**

The insole and stocking may be hand-washed in cold water with mild detergent and air-dried.

**FOR SINGLE PATIENT USE ONLY.**

~~NOT MADE WITH NATURAL RUBBER LATEX.~~

**RX ONLY.**

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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