ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Separates dislocated and closely positioned or overlapping big and second toes
- Helps to prevent pain from pressure or rubbing, skin irritation and corns between the big and second toes
- Helps to relieve pain and discomfort from bunions
- Helps to limit the deviation of the big toe (Hallux Valgus) by improving its alignment

Performance characteristics: Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

CONTRAINDICATIONS

- If you have the following conditions please consult a physician before using the device: gout attack, peripheral arterial disease or impaired blood circulation, rheumatic diseases, hypersensitivity to pressure, base joint arthritis, polyneuropathy or soft tissue swelling.
- Do not use this device if you have skin diseases or injuries in the affected area of the body.
- Any known allergies to the medical devices materials.

WARNING & PRECAUTIONS

The product is intended for single patient use only. The product must be worn directly on the skin. Do not apply any lotions or ointments on the relevant part of the body while wearing the support. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately. Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

Apply to clean and dry skin. Place between the big and second toes.

CLEANING INSTRUCTIONS

Hand wash in warm water with mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not rub. Powder if necessary.

COMPOSITION

SEBS (Styrene-ethylene-butylene-styrene), talcum powder, natural aroma of menthol.

CLEANING INSTRUCTIONS

Hand wash in warm water with mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not rub. Powder if necessary.

COMPOSITION

SEBS (Styrene-ethylene-butylene-styrene), talcum powder, natural aroma of menthol.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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