
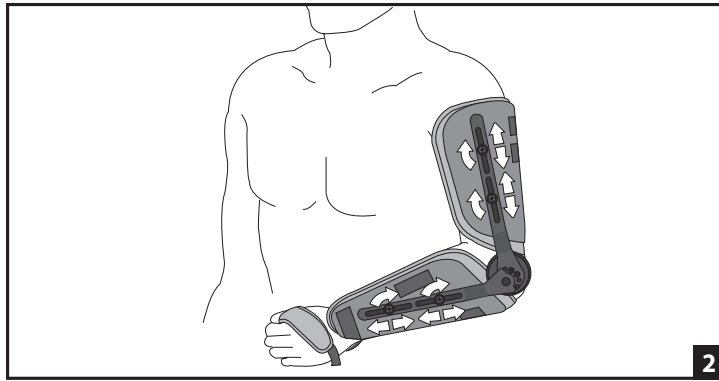
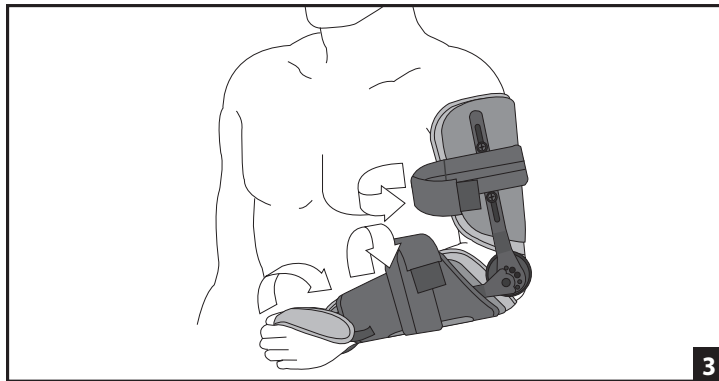
1


2

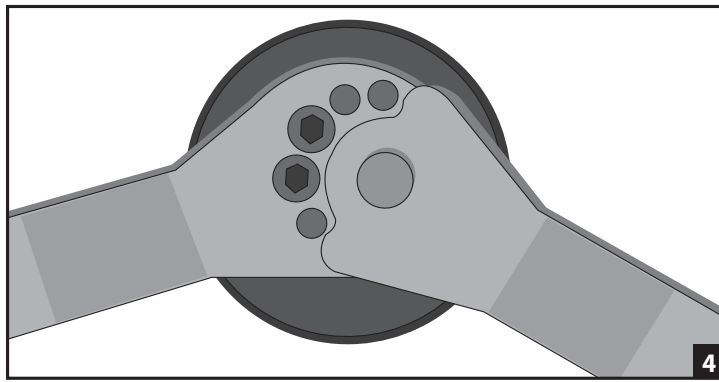3

4

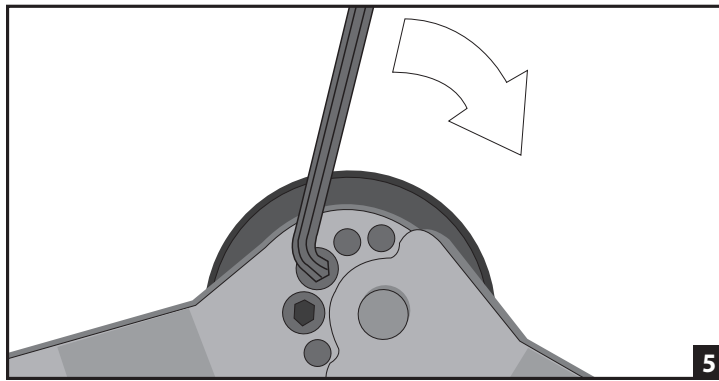5

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INDICATIONS: The Procure Elbow Ranger is designed to provide support and immobilization to the elbow. It may be suitable for use following stable radial head fractures. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS

This product should not be worn by individuals with known susceptibility to dermatitis or allergies to any of the materials contained within the product, particularly neoprene.

WARNINGS AND PRECAUTIONS:

- This product is to be prescribed and fitted by a healthcare professional.
- Do not use over open wounds.
- Excessive tightening of the brace straps may compromise circulation. If tingling or numbness is experienced, loosen straps.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.
- Do not use this device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1. Using accessory tool provided, loosen screws located in the slotted hinge bars so that the shells slide freely on the hinge bars. (Fig. 1)
2. Place arm into brace, positioning hinges even with the epicondyles and adjust (slide) the shells along the hinge bars to fit the correct arm length position. Tighten screws to secure shells to the hinge bar assembly. (Fig. 2)
3. To place hinge in a more anterior position, use accessory tool to remove screws located in the slotted hinge bars & place hinge bars into anterior shell holes. Replace and tighten screws to secure shells to the hinge bar assembly.
4. Secure forearm circumference wrap followed by the upper arm straps. (Fig. 3)
5. Support Sling Unroll sling. Attach one end of the sling across the humeral straps at a slight angle. Bring sling around neck and under forearm piece. Bring strap back on itself on the uninvolved side and secure.

HINGE ADJUSTMENT INSTRUCTIONS:

6. Hinge settings: Pin adjustments are set at 30° increments. (Fig. 4)
WARNING: To avoid damage to hinges and potential injury, both flexion settings must be identical and both extension settings must be identical.
7. To change ROM settings, use allen wrench provided to remove screws. Move to desired location and resecure. (Fig. 5)

CLEANING INSTRUCTIONS: Hand wash in water (86°F/30°C) using mild soap. Air dry.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations; the provisions of such local regulations will apply.

MATERIAL CONTENT: Neoprene, Nylon, Aluminum, Plastic, Lycra, Foam, Steel and Zinc

**Rx ONLY
FOR SINGLE PATIENT USE ONLY**

NOT MADE WITH NATURAL RUBBER LATEX

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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©2020 DJO, LLC