ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

#### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

#### INTENDED USE/INDICATIONS:

The DonJoy Immo 3V brace is designed to provide support and immobilization to the knee. It may be suitable for use following a knee injury or soft tissue injury. Providing immobilization or controlled movement of the limb or body segment.

#### CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained in this product.

#### WARNINGS AND PRECAUTIONS:

- It is recommended to have the device fitted by a suitably qualified healthcare professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

#### APPLICATION INSTRUCTIONS:

- Open up the closure straps.
  - Position the 3 panels so as to best cover the patient's leg, placing the patellar opening over the patella. Try the immobilizer over the patient's leg.
  - Close the immobilizer and tighten the closure straps. Tighten the straps according to the circumference of the thigh and calf, making sure they are neither too tight nor too loose.
- Note: Shaping the posterior stay helps improve the stability of the immobilizer.

#### CARE:

Hand wash in cold water with mild soap, rinse thoroughly. Air dry. Do not machine dry or bleach.

#### COMPOSITION:

Polyester, Polyamide, Polyurethane, Polypropylene, Aluminium.

**FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC
DJO FRANCE S.A.S.
DJO LLC
MDSS GmbH