ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INDICATIONS: For protocols involving immobilization in external rotation: • Glenohumeral dislocation/subluxation • Capsular Shifts • Use after posterior shoulder stabilizations • Postoperative Bankart procedures • Soft tissue strains/repairs • Post operative positioning after release of severe anterior capsule contracture

APPLICATION INSTRUCTIONS:

Step 1

1) Unfasten the Velcro straps at the forearm and wrist positions on the outside of the sling.

2) Insert arm into the sling, resting the cushion on your hip. Place your arms far back as possible in the sling and the thumb strap between the thumb and first finger.

3) Reattach both the forearm and wrist straps on top of the sling to secure the arm.

Step 2

Align cushion on the injured side at waist level with the ball (front) away from your body. Place the pillow so the line on top of the pillow is parallel to the front of your body. (If you stand at a table, the line should be parallel to the table.)

Step 3

A) Bring waist strap around from the back of cushion around your waist, inserting into and attaching to the front buckle.

B) Adjust waist strap for proper fit.

Step 4

Unfasten the shoulder strap buckle at the front of the cushion. Using your free arm, reach behind your body and slip your arm through the shoulder strap. This motion will be similar to putting on a backpack, with the wide strap on the clavicular area and the thin pad in the underarm area.

Step 5

Reattach the shoulder strap to buckle at front of cushion.

Step 6

Adjust all straps for adequate stabilization of injured shoulder.

USE AND CARE

- Hand wash in water (86°F/30°C) using mild soap. AIR DRY. NOTE: If not rinsed thoroughly, residual soap may cause skin irritation or deteriorate the material.
- Not made with natural rubber latex.
- This product is intended for use on a single patient.






Material content: Nylon, Polyester, Spandex, Polyurethane

WARNING: This product is to be used under the supervision of a medical professional, this device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations; the provisions of such local regulations will apply.

FOR SINGLE PATIENT USE ONLY

RX PRODUCT

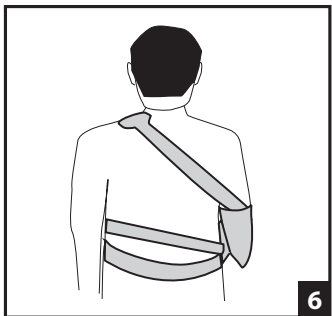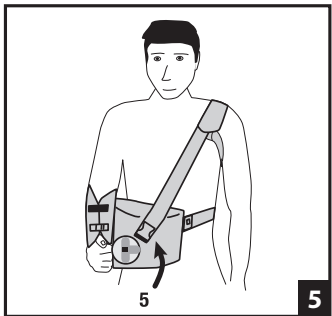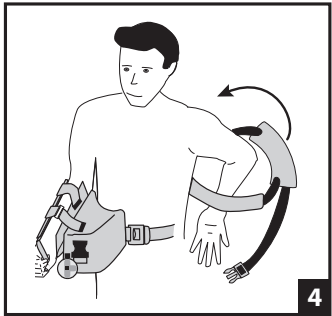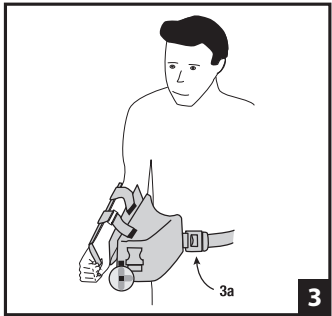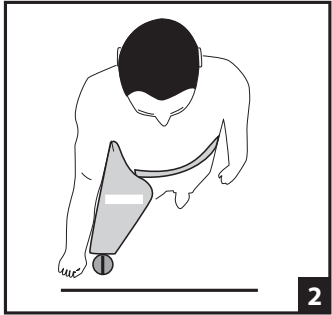CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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