ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance.

The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Helps to protect and relieve pressure on the forefoot area
- Provides support of the transverse arch, cushioning and comfort.

Performance characteristics: Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

CONTRAINDICATIONS

- If you have the following conditions please consult a physician before using the device: gout attack, peripheral arterial disease or impaired blood circulation, rheumatic diseases, hypersensitivity to pressure, base joint arthritis, polyneuropathy or soft tissue swelling.
- Do not use this device if you have skin diseases or injuries in the affected area of the body.
- Any known allergies to the medical devices materials.

WARNING & PRECAUTIONS

The product is intended for single patient use only. The product must be worn directly on the skin.

Do not apply any lotions or ointments on the relevant part of the body while wearing the support.

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

Apply to clean and dry skin. Secure the pad to the second or third toe with the ring. The letters differentiate between left (L) and right ® foot.

CLEANING INSTRUCTIONS

Hand wash in warm water with mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not rub. Powder if necessary.

COMPOSITION

SEBS (Styrene-ethylene-butylene-styrene), talcum powder, natural aroma of menthol

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