1

2

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: The Procure Squared-Toe Post-Op shoe is designed to provide support and protection to the foot. It may be suitable for use following surgery or trauma to the foot. Providing mild protection of a limb or body segment for acute and prophylactic care.

PERFORMANCE CHARACTERISTICS:

Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: None.

WARNINGS AND PRECAUTIONS:

This product is to be used under the supervision of a medical professional namely a physician and not intended for unsupervised public use.

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or packaging has been opened. If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS:

- 1) Unfasten ankle strap and forefoot closure. Slip foot into the shoe.
- 2) Secure both strap and forefoot closure for a snug but comfortable fit.

CLEANING INSTRUCTIONS:

To clean shoe, wipe with a damp cloth and air dry.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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