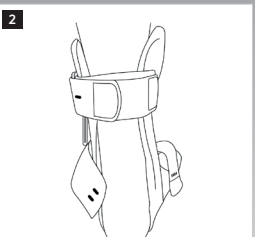
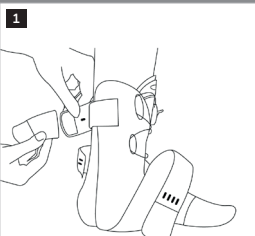
AIRCAST

AIRGO PLUS / AIRPRO

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ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all of the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

Optional extension strap for strap #1 of AIRGO Plus/AIRPRO. Soft-goods/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS & PRECAUTIONS

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- If you develop an allergic reaction and/or experience itching, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION - OPTIONAL EXTENSION STRAP

- To extend the calf strap, attach the extension strap to strap #1 of the AIRGO Plus/AIRPRO (Fig. 1)
- Pull the top strap around the back of the calf and fix to the external side of the brace without tightening too much (Fig. 2)
- Close the brace as usual.
- If the extension strap is too long, shorten it.

CARE

- Hand wash in lukewarm water (30° C) with a mild detergent.
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

COMPOSITION

Polyester, Polyamide

NOT MADE WITH

NATURAL RUBBER LATEX

FOR SINGLE PATIENT USE ONLY.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC