AIRCAST®

AIRGO PLUS / AIRSPORT +

STABILISATION STRAP

ZUSATZGURT

SANGLE DE STABILISATION

DEUTSCH

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ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The Aircast Stabilization Strap is designed to provide additional optional support and stabilization to the ankle when using the Airgo Plus or AirSport+ brace. Soft-goods/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION - OPTIONAL STABILISATION STRAP

- To increase stability, you can extend the front cross-strap of AIRGO Plus/AirSport+ by using the stabilisation strap. Attach it to the cross-strap, with the arrow pointing to the right direction. (Fig. A)
 - Wrap the extended strap, centered around your Achilles tendon. (Fig. B)
 - Bring the strap forward, crossing over your forefoot. (Fig. C)
 - Run the strap under your foot. (Fig. D)
 - E Fasten the strap across the front. (Fig. E)
- If the stabilisation strap is too long, shorten it.

CARE

- Hand wash in lukewarm water (30 °C) with a mild detergent
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

COMPOSITION

Polyester, Polyamide

FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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MDSS GmbH
DJO FRANCE S.A.S.